



## Turning and Repositioning System



### Indications for Use

- Assisting caregivers with patient turning and repositioning
- Prevention and management of pressure injuries
- Patient comfort
- Use across the continuum of care

### Precautions

- Always follow your facility's protocols when turning, repositioning or transferring patients
- Do not lift patients off the bed with the ATR+® System
- A minimum of two caregivers is recommended when using the ATR+ System
- Obtain medical clearance when turning patients with potential spinal instability (i.e. trauma, recent spine surgery)
- Single patient use
- Weight Limit: 575 lbs.

## Instructions for Use

### Set-Up

#### Step 1: Prepare



After locking the wheels in place, log roll the patient to one side of the support surface.

#### Step 2: Unfold Sheet



Unfold the Repositioning Sheet alongside the patient, with the product label on the back side of the sheet at the foot of the bed. Align the top of the sheet with the top of the patient's head.

#### Step 3: Roll & Repeat



Tuck the sheet under the patient and unfold it toward you. Log roll the patient to the opposite side and repeat this process. Return the patient to the supine position.

Continued on other side ►

# ATR+® System Additional Instructions for Use

## Turning and Repositioning

### Step 1: Position Patient



Grasp the ergonomic handles on the Repositioning Sheet. Hand off the handles to the caregiver on the opposite side while rolling the patient.

### Step 2: Place Wedges



Place the first Position Perfect® Wedge just above the waist area and the second just below the hip area. Wedges should be placed one hand-width apart and should not cross the patient's mid-line. Return the patient down onto the wedges.

### Step 3: Check



The wedges are properly positioned when you can slide your hand between them and verify the sacrum is offloaded.

## Patient Boosting

### Step 1: Prepare Surface



Ensure patient's support surface is at a comfortable height and locked in place.

### Step 2: Remove Wedges



Using just one of the wedge handles, gently remove from underneath the patient by rotating the wedge while pulling it out.

### Step 3: Boost Patient



Both caregivers grasp the ergonomic handles and boost the patient to the desired position.

## Cleaning Instructions

The ComfortTop® ES Absorption Pad should be disposed of after use and replaced with a new pad. Position Perfect Wedges and Repositioning Sheet are single patient use but can be cleaned with standard hospital cleaning wipes if soiled. A complete list of approved cleaning products are available upon request.

**For complete instructions, refer to product package insert.**



**Scan the QR code to view an instructional video.**



250 N Belmont Avenue • Indianapolis, IN 46222 • 800.899.5553 • **EHOB.com**

ATR+ is a trademark of EHOB, Inc. © 2023 EHOB, Inc. MKG002417 REV02 10-23