



Indications for Use

- For reducing the risk of injury to patients who require assistance with lateral transfers or repositioning
- For reducing the risk of musculoskeletal injury in health care workers during repositioning and lateral transfers of patients who require assistance
- For maintaining proper patient positioning in patients who are at risk of pressure injury
- For moisture management in patients who are at risk for moisture associated skin damage

Precautions

- Do not use with patient lifting devices or to suspend patient

Instructions for Use

Set-Up



Lock the wheels and lower head of bed to ensure it is a comfortable height for both caregivers. Lower the side rail and gently roll the patient onto their side.



Place BAM Pro vertically on the support surface with Reinforced Handles along the edge of the bed. Unfold, center and tuck BAM Pro under the patient. Raise the side rail.



Roll the patient to the opposite side and unfold the remaining edge of BAM Pro. Return the patient to the supine position. Ensure the patient is centered on the product.

Boost



Before you begin boosting, ensure the patient's bed is locked and at a comfortable height for caregivers.



Lower the head of bed and lower the side rail.



Each caregiver grasps the BAM Pro Reinforced Handles that feel most appropriate.

Assist



To place a patient in the lateral side-lying position, the caregiver on one side of the bed grasps the BAM Pro Reinforced Handles. Then, hand off the Reinforced Handles to the caregiver on the opposite side while rolling the patient.



Place the first Position Perfect Wedge just above the waist area and the second just below the hip area, ensuring they are not placed past the midline as you return the patient down onto them to offload the sacrum.



To remove the Position Perfect Wedges, use the orange Ergonomic Handles. Remove the first wedge just below the hip area and the second just above the waist area.

Move



Before you begin lateral patient transfers, ensure the patient's bed and adjoining surface are locked and at a comfortable height for all caregivers. Lower both side rails.



With caregivers at the side of the bed, and the others at the side of the adjoining surface, use the BAM Pro Reinforced Handles to transfer the patient onto the receiving surface.

Cleaning Instructions

BAM Pro Repositioning Sheet can be wiped clean. Discard Incontinence Pad when soiled.

Learn More ▶

