


BAM Sheet™

Boost • Assist • Move

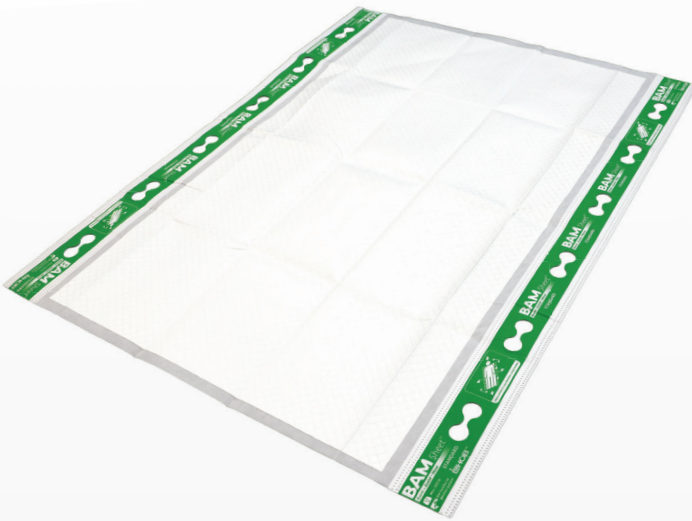

Indications for Use

- Aids with repositioning and lateral transfers for people who require assistance
- Aids in optimizing microclimate for people at risk of skin damage due to moisture
- Aids in containing fluids to protect underlying surfaces

Precautions

- Do not use with patient lifting devices
- Do not use to suspend patient

Follow your facility protocols for patient handling, turning, and repositioning

Instructions for Use

Set-Up

Step 1



Lock the wheels and lower head of bed to ensure it is at a comfortable waist height. Lower both side rails and gently roll the patient onto their side. Place BAM™ Sheet vertically, with the Reinforced Handles along the edge of bed. Unfold, center and tuck BAM Sheet under the patient.

Step 2



Roll the patient to the opposite side and unfold the remaining edge.

Step 3



Return the patient to the supine position. Ensure the patient is centered on the product.

Continued on other side ►

BAM™ Sheet Product Education Continued

Boost



Ensure the patient's bed is locked and at a comfortable waist height.



Lower head of bed and lower both side rails.



To facilitate optimal ergonomic support, each caregiver grasps the Reinforced Handles, that feel most appropriate for boosting.

Assist



Ensure the patient's bed is locked and at a comfortable waist height. Lower the head of bed and lower both side rails. To place a patient in a lateral side-lying position, the caregiver on one side of the bed grasps the Reinforced Handles. He or she hands off the Reinforced Handles to the caregiver on the opposite side while rolling the patient.



Place the 1st Position Perfect Covered Wedge just above the waist area and the 2nd just below the hip area to offload the sacrum. Return the patient down onto the wedges. The wedges are properly positioned when you can slide your hand between them and verify the sacrum is offloaded.



To remove the wedges, use the Ergonomic Handles to first remove the wedge just below the hip area, and the second just above the waist area.

Move



Ensure the patient's bed and adjoining surface are locked and at a comfortable waist height for both caregivers. Lower both side rails.



With one caregiver at the side of the bed, and another at the side of the adjoining surface, use the BAM Sheet Reinforced Handles to transfer the patient onto the receiving surface.



Ensure the patient is centered on the product.

Cleaning Instructions

BAM Sheet should be disposed of after use and replaced with a new sheet.

For complete instructions, refer to product package insert.

Scan the QR code to learn more. ▶



250 N Belmont Avenue • Indianapolis, IN 46222 • 800.899.5553 • **EHOB.com**