


Product Selection Decision Tree

Sample Risk Assessment Indicators:

- Braden Mobility sub score of 1-3
- Braden Sensory Perception sub score of 1-3
- For patients unable to lift heel off of bed independently
- For patients unable to reposition themselves independently
- For patients with existing or a history of heel pressure injuries
- For patients with the medical condition of foot drop
- For patients at risk for ankle contractures

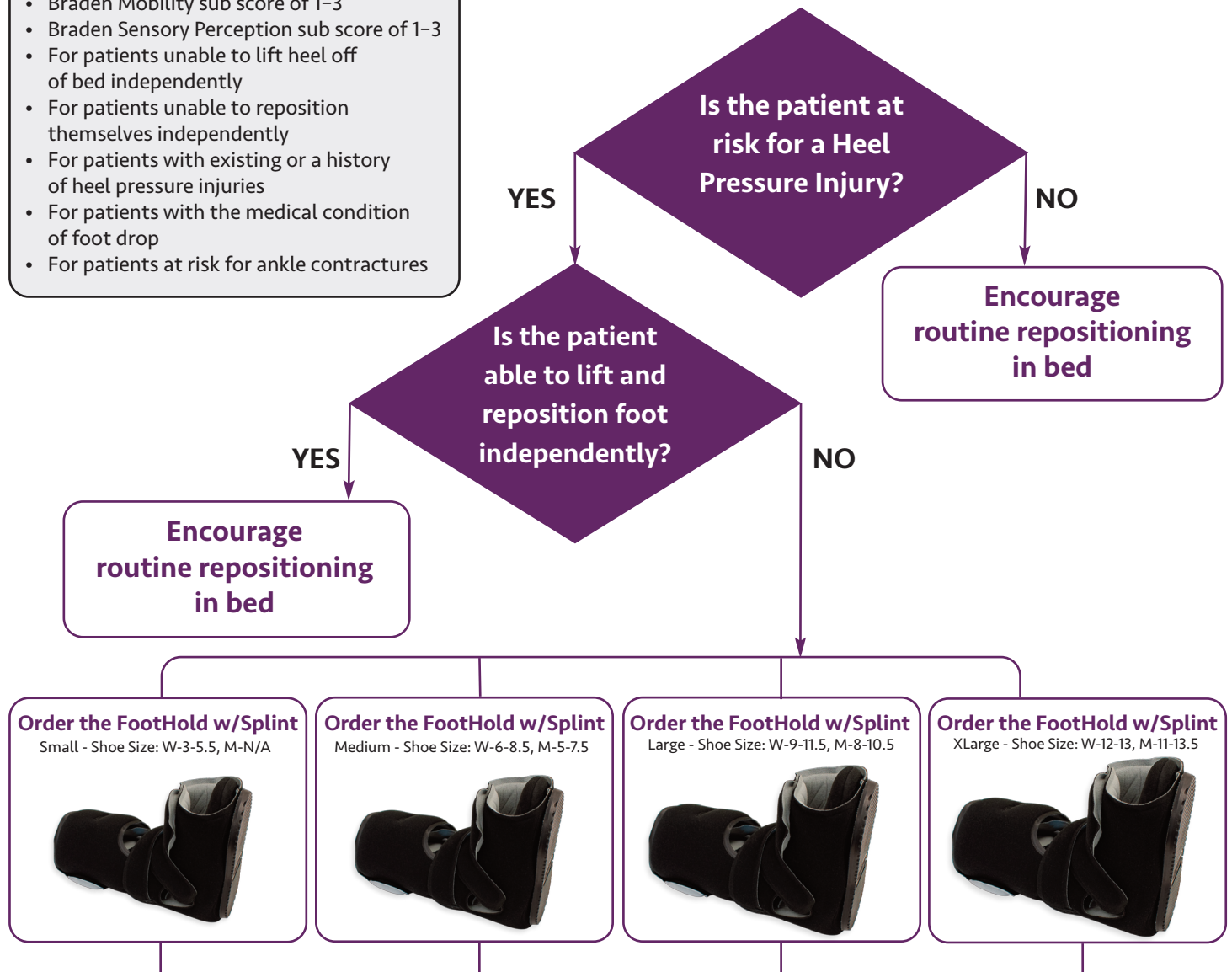

- Remove each shift and as needed for skin assessment
- Re-adjust to ensure proper placement every shift and after repositioning
- Ensure Secure Stick Sole is in place for chair transfers and ambulation
- Ensure tubing is directed away from patient's skin
- Remove Secure Stick Sole while in bed
- When a single boot is in use, recommend wearing a shoe on opposite limb during ambulation to reduce height difference
- The FootHold with Splint can be used across the continuum of care
- Single patient use