

Product Selection Decision Tree

Sample Risk Assessment Indicators:

- Limited ability to reposition self
- Braden score of 18 or less
- Braden Activity subscore of 1-3
- Braden Mobility subscore of 1 or 2
- Braden Friction and Shear subscore of 1 or 2
- Existing or history of pressure injuries

Is the patient at risk for a Heel Pressure Injury?

YES

NO

Is the patient able to lift and reposition foot independently?

YES

NO

Encourage routine repositioning in bed

Encourage routine repositioning in bed


Order the Foot WAFFLE® - small
11.5" - 13" calf circumference



Order the Foot WAFFLE® - medium
13" - 18" calf circumference



Order the Foot WAFFLE® - large
18" - 21.5" calf circumference

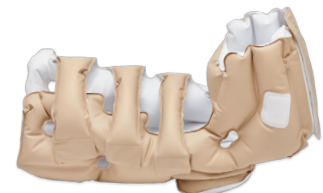

- Remove each shift and as needed for skin assessment
- Re-adjust to ensure proper placement every shift and after repositioning
- Remove for chair transfers and ambulation
- Utilize tubing port to direct tubing away from patient's skin
- The Foot WAFFLE can be used across the continuum of care
- Single patient use