


Product Selection Decision Tree

Sample Risk Assessment Indicators:

- Limited ability to reposition self
- Braden score of 18 or less
- Braden Moisture subscore of 1 or 2
- Braden Mobility subscore of 1 or 2
- Existing or history of pressure injuries

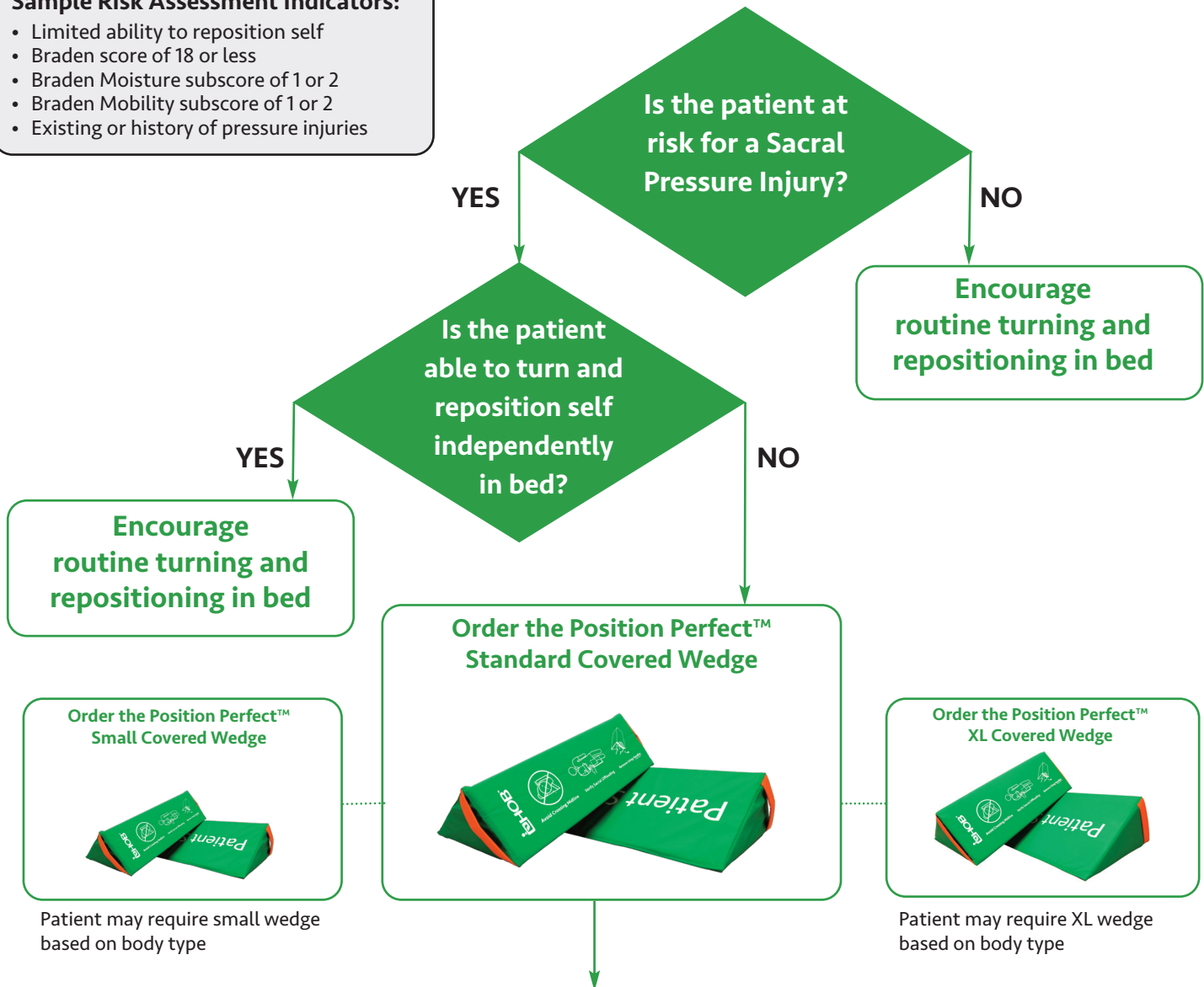

- Wedges are properly positioned when you can slide your hand between them and verify the sacrum is offloaded
- Turn and reposition the patient every 2 hours or per individualized turning schedule
- Perform a hand check after each turn to confirm that the sacral area is offloaded
- Position Perfect Uncovered wedges available
- Send product(s) with patient at discharge