

Clinical Guidelines

Intended Use:

Assists in lateral transfers and maintaining proper patient positioning while managing moisture and reducing the risk of musculoskeletal injuries in health care workers.

Consider for patients with any of the following risk factors:

- Limited ability to reposition self
- Braden score of 18 or less
- Braden Moisture subscore of 1 or 2
- Braden Mobility subscore of 1 or 2
- Braden Friction and Shear subscore of 1 or 2
- Existing or history of pressure injuries

Indications for Use:

- For reducing the risk of injury to patients who require assistance with lateral transfers or repositioning
- For reducing the risk of musculoskeletal injury in health care workers during repositioning and lateral transfers of patients who require assistance
- For maintaining proper patient positioning in patients who are at risk of pressure injury
- For moisture management in patients who are at risk for moisture associated skin damage



Intended Care Setting: Intended to be used in hospitals and other inpatient healthcare facilities.

Suggested uses of the BAM Pro System:

Repositioning and Lateral Transfers

Consider for patients with limited mobility due to conditions such as:

- Acute illness
- Post-operative limitations or restrictions
- Intra-procedural limitations or restrictions
- Chronic illness with or without multiple comorbidities
- Musculoskeletal disorders
- Neurological deficits
- Orthopedic injuries

Moisture Control

Consider for patients with fluid management needs due to conditions such as:

- Incontinence
- Diaphoresis
- Wound drainage
- Surgical drains
- Traumatic injury

Consider when surface protection from excessive fluids is needed.

- Emergency Department/Trauma
- Labor and Delivery
- Surgery

Interventions

- Patient should be centered on product with handles along the edge of the bed with side stripes facing up
- A minimum of two caregivers is recommended when using the BAM Pro System
- Support head and legs during lateral transfers, if needed
- Follow facility's safe patient handling protocol when repositioning and laterally transferring patients and when placing and using an incontinence pad
- Use only one BAM Pro System at a time underneath the patient

Precaution

- Do not use with patient lifting devices
- Do not use to suspend patient

Additional Information

- Single patient use
- BAM Pro sheet can be wiped clean. Discard incontinence pad when soiled
- Patient may require the **BAM Pro XL** based on body type
- Use the ATR+/ATR+ Pro Repositioning System for patients who require longer hospital stays